

Veterinarians

How to connect a AFX-110 microchip scanner to the FEI HorseApp

You have a AFX-110 microchip scanner?

You will use the FEI HorseApp at your next Event to perform the Examination on Arrival?

Below you will find the step-by-step procedure to connect your scanner to the FEI HorseApp

Step 1: Turn on the scanner and press the right-hand button to bring up the main menu:

- scroll down to the **Bluetooth** settings
- press the left-hand button
- press the right-hand button to scroll down to **Select device** -> press the left-hand button

- press the right-hand button to scroll down to **Slave** -> press the left-hand button



The scanner will return to the main page with the **Bluetooth** sign flashing.

Step 2: On your mobile device, go to **Settings** -> **Bluetooth** and make sure the Bluetooth is ON on your device

iOS devices (iPhones & iPads):

Android devices:

Veterinarians

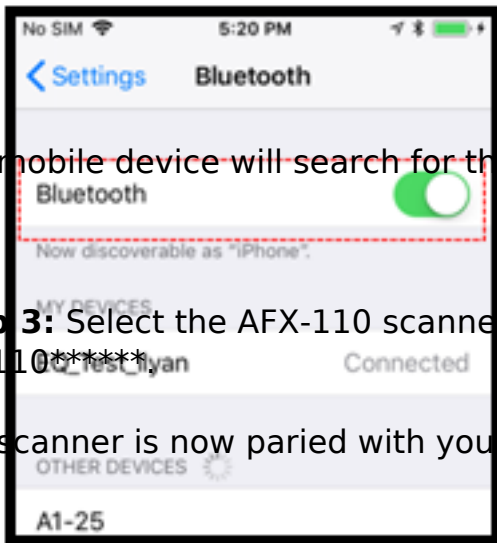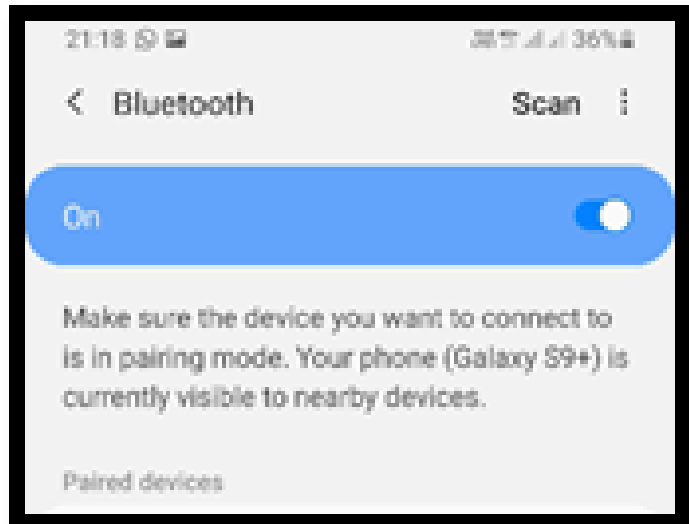

The mobile device will search for the microchip reader.


Step 3: Select the AFX-110 scanner that appears in the list under the name AFX110*****

The scanner is now paired with your device.




Step 4: Open the [FEI HorseApp](#) and login with your FEI credentials. Choose that you wish to **act as a Veterinarian**.


Click on **Not Connected** to connect the AFX-110 scanner to the FEI HorseApp.

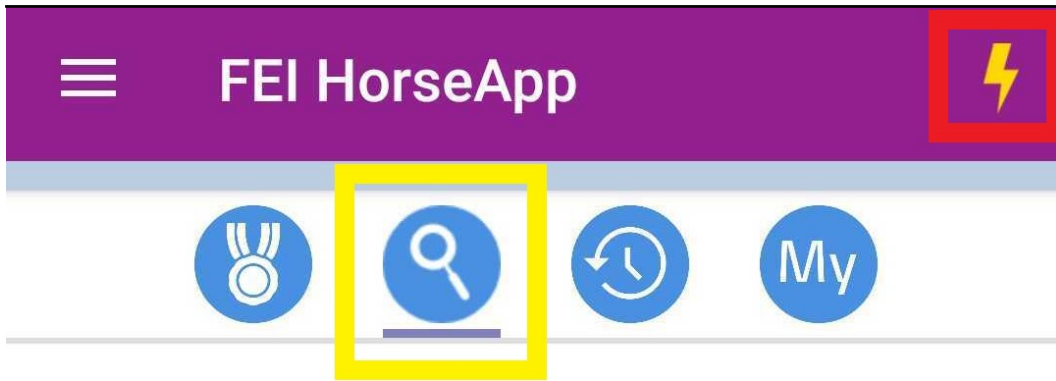Veterinarians




Horse Search

-  **Advanced search** >
Search for a horse by name or FEI ID
-  **Scan barcode** >
Scan a FEI document barcode
-  **Scan microchip** >
Scan with an external reader

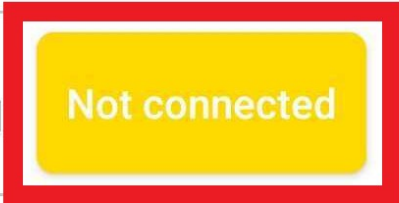

Veterinarians


Step 5: Locate the GPR+ scanner in the list and click on Connect.

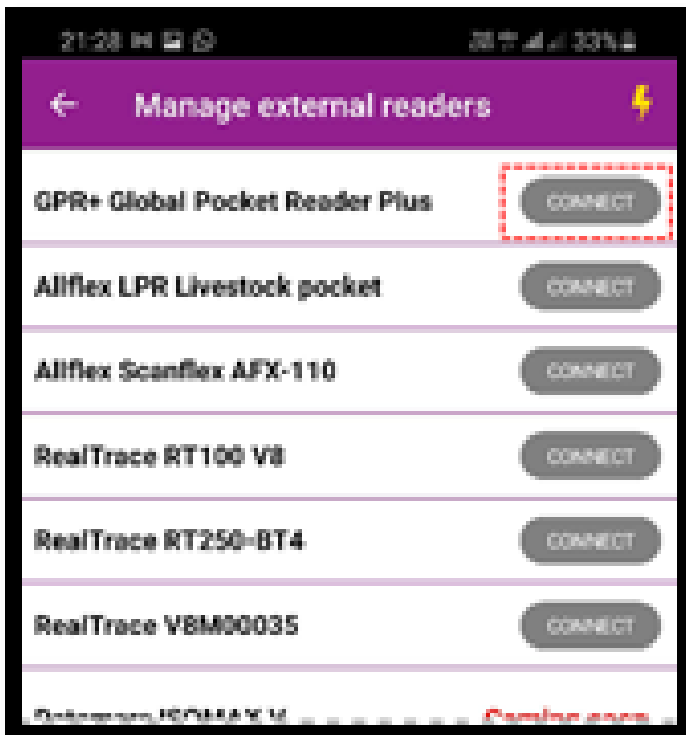

Please note: the first time you connect your GPR+ scanner it might take several minutes or attempts to connect it

Step 6: Once the GPR+ scanner is connected to the FEI HorseApp, the button will turn **green** and the **thunderbolt** logo will also show that you are correctly connected.

Veterinarians


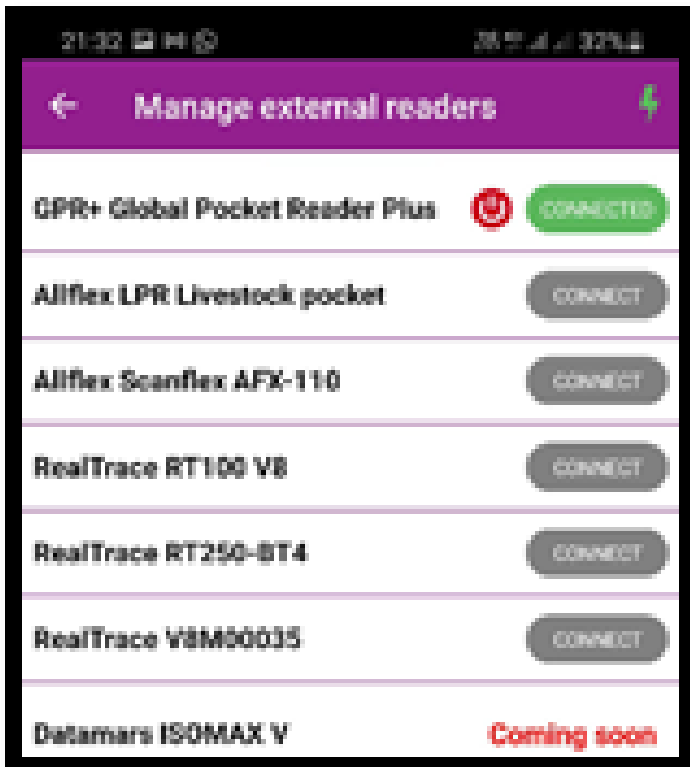


You can use this screen to disconnect your microchip scanner if you wish.

Step 7: You can now proceed with scanning a Horse. The scanner will work whenever it is **connected**.

To scan, search for the horse microchip and press the left-hand button on the AFX-110 scanner. The horse' profile will open on the FEI HorseApp accordingly.

Veterinarians

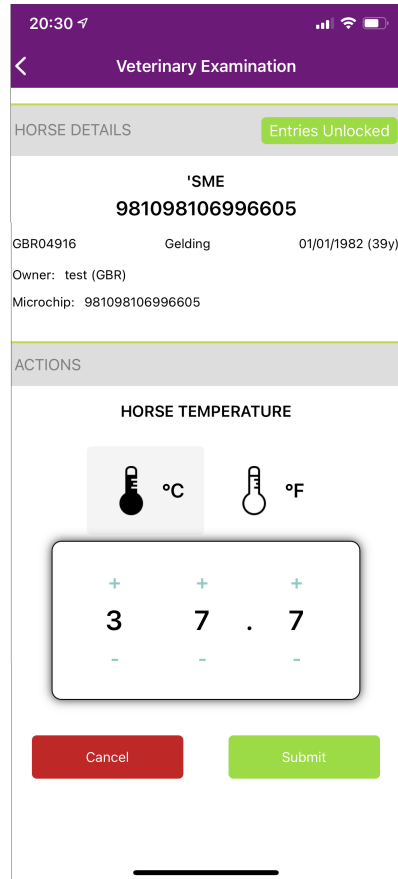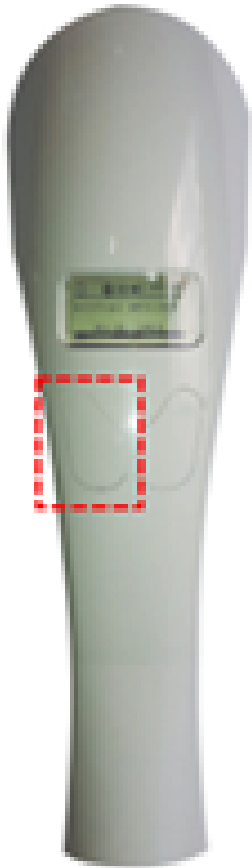

[Click here to see how to use the FEI HorseApp during the Examination on Arrival](#)

Unique solution ID: #1252
Author:
Last update: 2022-06-22 10:23